

# Comfort Flex-Tabs

## WHAT ARE THEY?

Comfort Flex-Tabs are designed to adhere to the non-woven (cloth-like) material of our breathable briefs. They are not tape tabs (sticky tabs); instead they use a hook-and-loop type refastening system so that they can be fastened again and again, and can be fastened anywhere on the brief. They are also significantly softer and more flexible than competitors' hook-and-loop tabs.



## HOW DO I USE THEM?

Hook-and-loop tabs need to be properly adhered to the brief if you are to achieve a strong attachment. For this reason, it is important to remember the “**PRESS and TUG**” technique:

Once you adhere the tab to the brief, make sure to **PRESS** it down firmly against the brief...

**THEN...**

**TUG** it gently from the side to make sure the hooks fully engage to the brief.

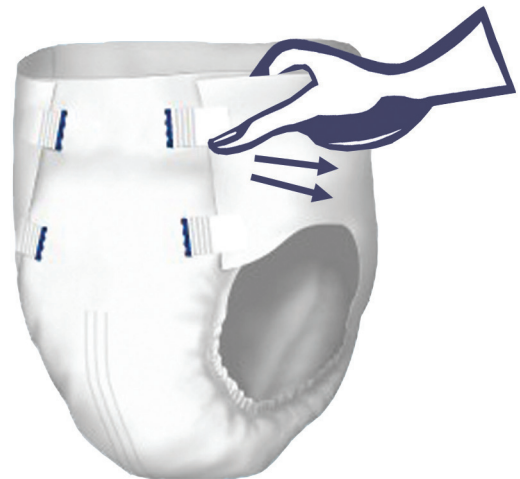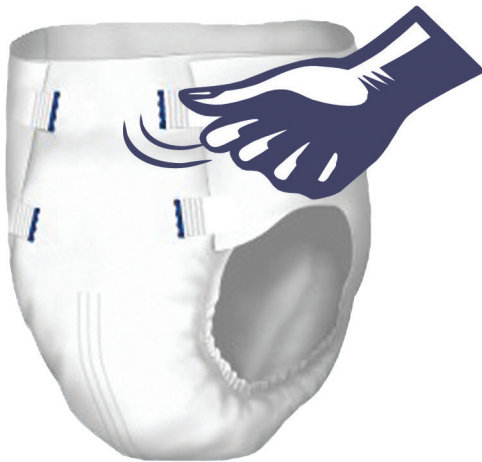

The “**PRESS and TUG**” technique will ensure that the hooks of the tab have engaged with the loops of the backsheet material.

Remember, as with regular tape tabs, angling the tabs can be helpful in customizing the fit to the resident's body type.